


TISSUE TRANSFER

HARVEST, WASH, TRANSFER and RE-INJECT Tissue with the Completely Contained, Closed Loop, A.T.T.S. Tissue Collection system.

A.T.T.S.

The A.T.T.S., Automatic Tissue Transfer system, is a tissue collection system made of a medical grade, sterilizable, implosion proof polymer with a tapered base and stainless steel drain port. From the point the adipose tissue is harvested from the patient to the point it is re-injected into the patient, the adipose tissue is completely contained in the A.T.T.S. collection system.

As the tissue is harvested from the patient it enters the collection system. A sterile normal saline solution is hooked up to the collection system via an IV bag and sterile tubing. The physician has the option to release the saline solution as the tissue enters the collection system or release the saline solution after all the tissue is contained in the collection canister or both. This process works to wash the adipose tissue while allowing all the impurities (tumescent solution, blood, oils, etc...) to sink to the base of the collection system leaving the clean good re-injectable tissue floating above. The physician is then able to remove the impurities from the collection system via the drain port at the base. This process leaves adipose tissue that can now be safely transferred to syringes by connecting a large syringe to the bottom drain port.

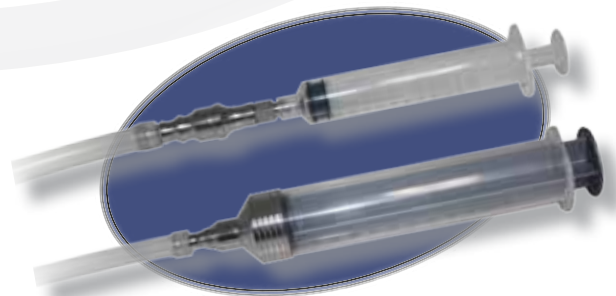

Connect to a Luer Lock syringe or
Toomey syringe for re-injection.

INNOVATIVE MED INC.
www.imibeauty.com



A.T.T.S.

The **A.T.T.S.** Complete system includes:

- 2 ATTS Canisters
- 1 ATTS Lid
- 1 ATTS Syringe Rack
- 1 ATTS Barbed Luer
- 1 ATTS Luer Transfer
- 1 ATTS Female Cap (6/pk)
- 1 ATTS Toomy Syringe Cap (6/pk)
- 1 ATTS Valve
- 1 ATTS 12" Silicone Tube
- 1 ATTS Mount System (Pole, Base, or Wall Mount)
- 1 ATTS Male Luer Cap
- 1 ATTS O-Ring (2/pk)
- 1 ATTS Valve O-Ring (2-2/pk)
- 1 ATTS Cleaning Brush
- 1 ATTS Wrench
- 1 ATTS Barb Cap



<u>Item No.</u>	<u>Description</u>
ATTS-CS	A.T.T.S. Complete System
ATTS-C2LD	A.T.T.S. Canister and Lid
ATTS-C2	A.T.T.S. Canister Only High Temp
ATTS-CLD	A.T.T.S. Canister Lid w/ Suction Barb
ATTS-SR126	A.T.T.S. Syringe Rack
ATTS-BL	A.T.T.S. Barbed Luer
ATTS-LTL	A.T.T.S. Luer Transfer
ATTS-FLC	A.T.T.S. Female Cap (6/pk)
ATTS-VAL	A.T.T.S. Valve for Canister
ATTS-12ST	A.T.T.S. 12" Silicone Tube
ATTS-BAS	A.T.T.S. Base Assembly
ATTS-MLC	A.T.T.S. Male Luer Cap
ATTS-ORL	A.T.T.S. O-Ring Replacement for Lid, pk/2 A.T.T.S. O-Ring for Valve
ATTS-VOC	Canister to Valve, pk/2
ATTS-VOV	Valve to Valve, pk/2
ATTS-TSB	A.T.T.S. Toomy Syringe to Barb Hub
ATTS-TOC	A.T.T.S. Toomy Syringe Cap (6/pk)
ATTS-BRU	A.T.T.S. Cleaning Brush
ATTS-WR-V	A.T.T.S. Wrench for Valve, stainless steel
ATTS-BC	A.T.T.S. Barb Cap for Canister Lid

<u>Item No.</u>	<u>Description</u>
ATTS-FTN	Facial Tissue Transfer Needle, Style I, 16ga X 9cm luer lock Hub
ATTS-BTN	Body Tissue Transfer Needle, Style I, 3mm X 23cm Toomy Hub
ATTS-PMS	A.T.T.S. Pole Mount System
ATTS-VAS-M	A.T.T.S. Wall Mount (Fits Vasor & Medela)
ATTS-LDV	A.T.T.S. Canister Lid Vented w/ Luer Caps

INNOVATIVE MED INC.
www.imibeauty.com

4 Autry, Suite B • Irvine, CA 92618 • U.S.A.
 949.458.1897 • 877.779.9492